INSTRUCTIONS FOR USE

- **REF** REFERENCE: CP614, CP614-SB, CP614-DK-1, CP614-DK-2, CP614-DK-3, CP634, CP634-SB, CP634-DK-5, CP778, CP778-SB, CP778-DK-1, CP778-DK-2, CP778-DK-3, CP471S, CP471S-SB, CP471S-DK-2, CP471S-DK-3, CP471S-DK-4, CP471S-DK-5, CP471D1, CP471D1-SB, CP471D1-P-1S/L, CP471D1-P-3S/L, CP1038S, CP1038S-SB, CP1038D1, CP1038D1-SB, CP1038D1-P-1S/L, CP1038D1-P-3S/L, CP1038D2, CP1038D2-SB, CP1038D1-NM, CP1038D1-NM-SB, CP1038D2-NM, CP1038S-NM, CP1038S-NM-SB, CP614-NM, CP614-DK-SB, CP634-NM, CP778-DK-SB, E7406, E7407, E7409, E7411
- The SteriBest Instrument Tray and mat should be cleaned prior to each use in accordance with the below cleaning instructions.
- Intended Users: Persons who have been trained in sterilizing instruments.
- SteriBest instrument trays are designed to encase non-lumened instruments during sterilization.
-  Always inspect for cleanliness or damage before each use. Do not use if damaged.
- Configure the tray, mat, and inserts (if applicable) properly to accommodate the instruments that are meant to be sterilized. The mat should be used to help stabilize, separate, and protect the instruments while inside the tray. Instruments should be arranged in a single layer with adequate spacing to prevent them from coming into contact with each other.
- Do not overload instrument trays or block/obstruct ventilation holes.
- An appropriate validated sterile barrier is required to maintain sterility, after sterilization and before opening of the pack.
- An approved sterility assurance device (i.e. Multi-parameter chemical indicator) should be placed inside and on each level (when applicable).
- Cover the tray with the lid and press down until lid is secure.
- The tray(s) should always be placed on the sterilizer rack or pouch/peg rack prior to sterilization so that air or steam circulation is not impeded, and so that they are not touching other trays or racks. Additional dry time may be necessary when using pouch/peg racks because moisture may pool on the corner/side.
- Stacking of trays inside the autoclave chamber is not recommended.
- Use of SteriBest instrument trays must be in accordance with the operating instructions supplied with your particular sterilization equipment and in keeping with your particular sterilization equipment and with your hospital policy on sterilization validation. Several organizations (e.g., AAMI, AORN, and ISO) publish guidance information on sterilization usage should you require additional information.
- Sterilize the loaded tray(s) in accordance with the instrument device manufacturer's guidelines for Pre-Vacuum steam sterilization.
- SteriBest instrument trays have been validated for use with four minute 132 degrees Centigrade pre-vacuum steam sterilization cycle. Copy of the validation report is available at <https://www.ricasurgical.com/catalogs-resources.asp>
- Do not use dry heat sterilization methods.

Cleaning Instructions

- After each use, all components of the SteriBest Instrument Tray should be washed in a mild, neutral pH enzymatic detergent. For optimal removal of soil and debris, soft sponges and soft cloths may be used to clean the plastic tray components. Hard to reach spaces such as between the pins of the mat may be cleaned using a soft bristled

RICA Surgical Products, Inc.

Phone: 1-800-889-3218 | Fax: 1-847-671-5690 | www.ricasurgical.com
9207 Ivanhoe Street | Schiller Park, IL 60176

brush. Abrasive cleaning chemicals, brushes, or pads should not be used.

- After cleaning with detergent, the tray should be washed thoroughly with warm distilled, reverse-osmosis, or comparable water for at least one minute. A clean, soft, absorbent, non-shedding towel may be used to dry the tray. The tray may be air-dried. Inspections of components should happen after each cleaning in order to verify cleanliness, lack of damage, and proper functionality.

Manufacturer:
RICA Surgical Products, Inc., 9207 Ivanhoe Street, Schiller Park, IL 60176

Authorized EC Representative:
B. M. Sanitas S.R.L., Via Caduti di Cefalonia 90, 50127 Florence, Italy

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Fabbricante:

RICA Surgical Products, Inc., 9207 Ivanhoe Street, Schiller Park, IL 60176

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B. M. Sanitas S.R.L., Via Caduti di Cefalonia 90, 50127 Florence, Italy

RICA Surgical Products, Inc.

Phone: 1-800-889-3218 | Fax: 1-847-671-5690 | www.ricasurgical.com
9207 Ivanhoe Street | Schiller Park, IL 60176